Vink

makes life just a little bit easier

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EN **Preface:**

VINK Headlock for Cows With the VINK Headlock for Cows the cow's head can be lifted and locked.

Difficult treatments such as administering medicine or putting in ear-tags, can be done by one person.

The Headlock is constructed of steel tubes and a formed plate and is used with a self-locking feeding barrier.

The more often a cow is treated the more difficult the cow becomes, but with this apparatus the treatment is easily performed. For the administration of medicine one has both hands free and there is no danger for man or animal.

Safety instructions:

The apparatus is only to be used by professionals and only with milking cows. The apparatus is developed for the treatment of milkcows and should only be used for these animals

The use of the headlock for cows:

Fasten the cow in the self-locking feeding barrier. Place the U-shaped frame from below upwards under the throat of the animal. If the animal does not cooperate, do not use force but calmly keep on trying.

Hang the apparatus on the upper tube of the feeding barrier. Lift the cow's head in the required position and fasten the support with the chain

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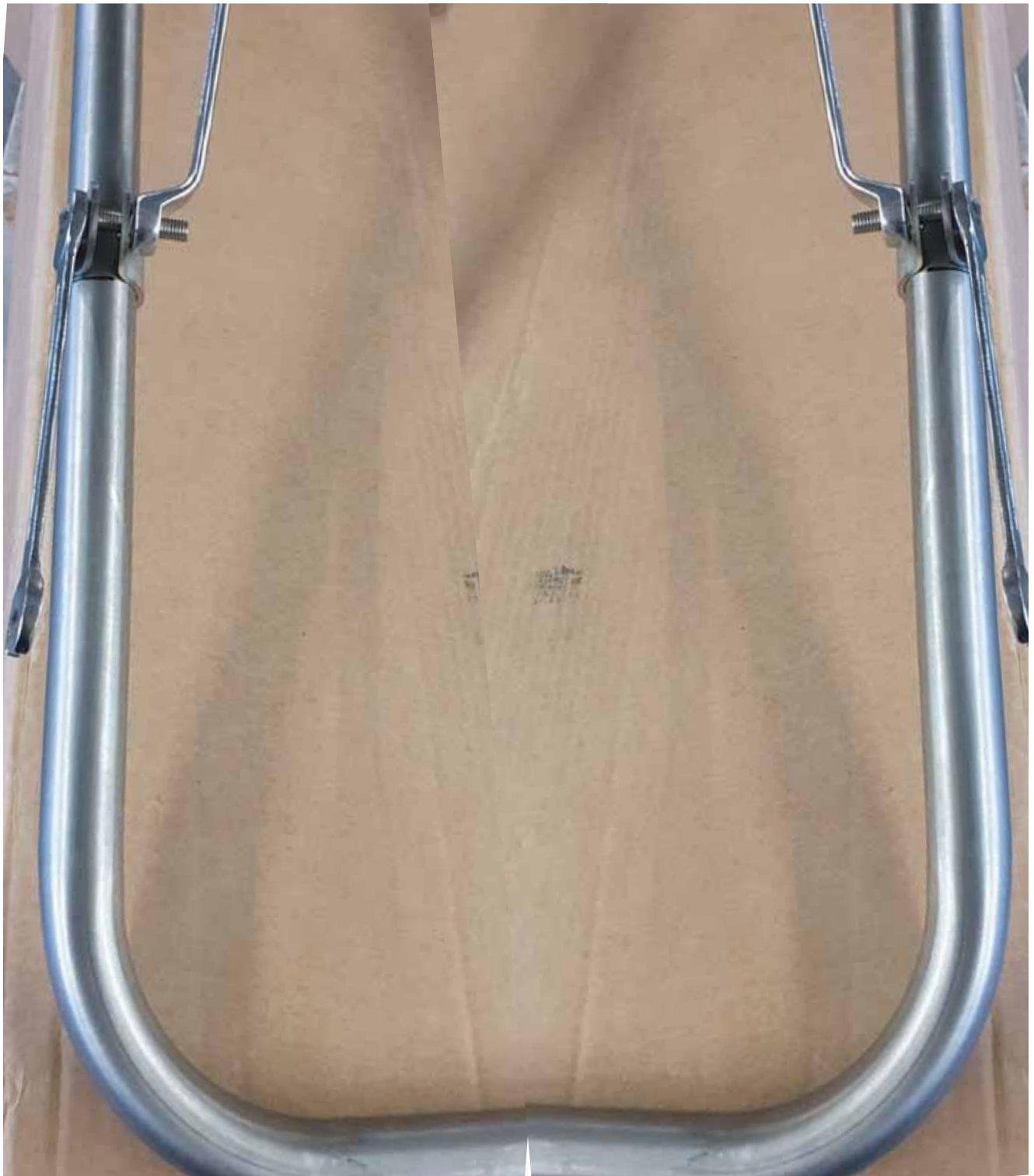
Slide the hinge brackets onto
the frame one left two right

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Screw the bracket firmly
fasten with screws m8x35 and the normal nut

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First slide the headrest over the bolt on one side

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And then with light pressure on the other side

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Now tighten the locknut on both sides just barely through the security

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Attach the chain on the right side m6x30 with lock nut

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Website: www.vink-elst.nl

We declare that, VINK Headlock

conforms to the Machine Directives 2006/42/EG as recently amended.

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Hersteller: Vink-Elst B.V.
Manufacturer: Vink-Elst B.V.
Fabrikant: Vink Elst B.V.,